## ENGLISH

### DESCRIPTION

FenuHoney Hydrocolloid is a product consisting of medical-grade Buckwheat Honey with Fenugreek Extract for the management of wounds and burns.

### INTENDED USE

FenuHoney with active Buckwheat Honey are intended for the management of light to moderately exuding wounds by providing a moist environment, which is conducive to wound healing, and supports and aids autolytic debridement. These dressings are intended for the management of the following:

- Diabetic foot ulcers
- Leg ulcers (venous stasis ulcers, arterial ulcers and leg ulcers of mixed etiology)
- Pressure ulcers/sores (partial and full thickness)
- 1st and 2nd degree partial thickness burns
- Donor sites
- Traumatic and surgical wounds

**CAUTION:** Federal (U.S.A.) law restricts that this device to sale by or on the order of a licensed healthcare practitioner.

### DIRECTIONS FOR USE

The FenuHoney Hydrocolloid Dressing has protective liners on either side of the dressing. Remove these prior to use. Place the dressing onto the wound surface ensuring that the dressing is in contact with the wound bed. Use a barrier cream as needed.

The dressing may be cut with sterile scissors or folded to fit the shape of the wound.

A suitable secondary sterile dressing should be applied to cover the FenuHoney dressing. The secondary dressing should sufficiently absorb any excess wound exudate. For heavier exuding wounds, fenestrate the FenuHoney dressing to allow the exudate to drain through to the secondary dressing.

Clean the wound between dressing changes using standard wound cleaning protocol. Suitable for use under compression dressings.

### FREQUENCY OF CHANGE

FenuHoney dressings require changing when they have been diluted by wound exudate and/or when the dressing has gelled (a white or opaque area will form). The dressing may be left in place for up to 7 days depending on the amount of wound exudate. Saturated dressings should be changed.

FenuHoney Hydrocolloid Dressing is non-adherent. It is easy to remove without disrupting the wound bed and new tissue.

Heavily exuding wounds require close monitoring and may require daily dressing changes.

### CONTRAINDICATIONS

Do not use on individuals who have a known sensitivity to any component present in the dressing. This device is not indicated for use in third degree burns.

### PRECAUTIONS

Transient stinging may be experienced on application of the dressing. This is due to the natural properties of the honey and is not detrimental to the wound. If transient stinging

persists and cannot be managed with an analgesic after 30 minutes, remove the dressing and wash the affected area.

When used on dry wounds, moisten the dressing with sterile saline or water to aid removal. Use with caution on heavily bleeding wounds.

The dressings high osmotic potential may contribute to increased exudate on initial treatment (depending on amount of interstitial fluid or edema). This can lead to some maceration if not managed properly with secondary absorbent dressings or more frequent dressing changes initially.

### STORAGE

Store at room temperature. Avoid excessive heat or humidity. Do not refrigerate.

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Manufactured for : Bennett Health, Inc,